## PRODUCT INFORMATION & FIT INSTRUCTIONS

mute

BREATHE MORE  
SNORE LESS  
SLEEP BETTER

[MUTESNORING.COM](http://MUTESNORING.COM)


## INSTRUCTIONS


EN

Mute™ is a respiratory technology designed to gently open your nasal airway during sleep to increase airflow through your nose and reduce mouth breathing. By breathing more freely through the nose, the Mute may reduce the effects of night time nasal congestion and may reduce snoring. It is made from a soft medical grade polymer and is safe for use in the nose during sleep. Use Mute only as directed. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the member state in which the user/and or patient is established.

- Mute has two soft paddles designed to gently expand your airway. They are adjustable for optimal dilation, comfort and fit.
- By moving the pin through the ratchet, the paddles move inwards for less dilation and outwards for greater expansion of the airways.
- The bridge connects the two parts together and is the only part visible outside your nose when fitted.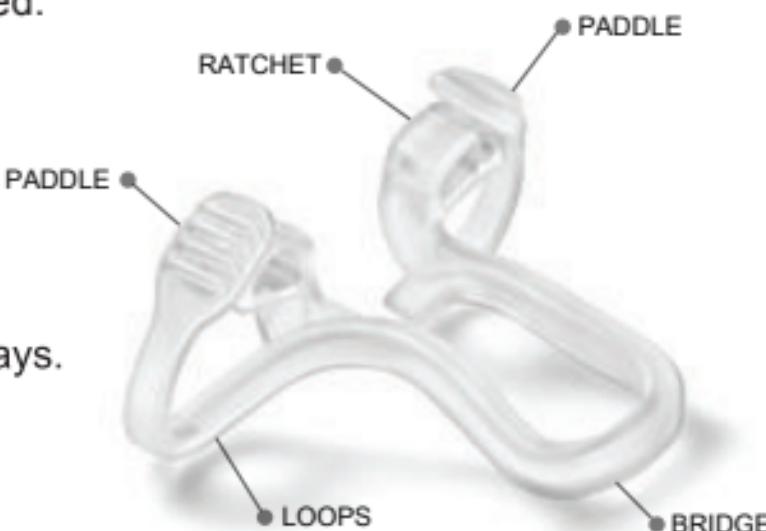

1

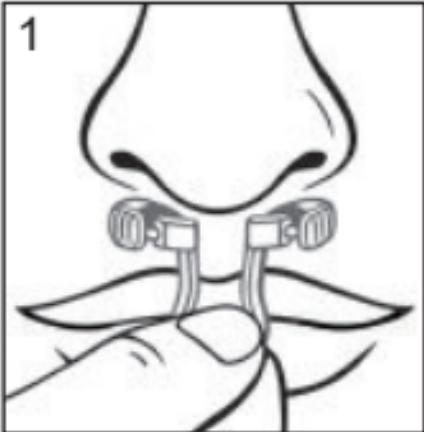
EN

Wash your hands before removing the product from its packaging.

Hold the bridge with the loops facing toward you and the paddles facing up and away from you.

2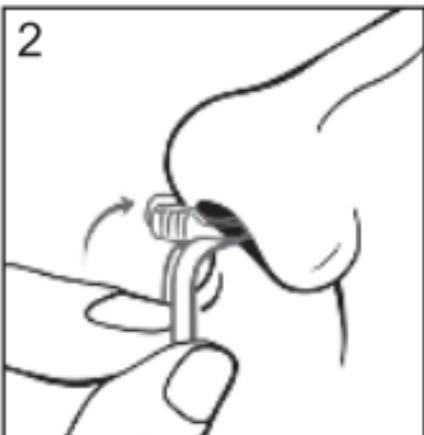

Gently rotate Mute into your nose by placing the loops in first whilst tilting the bridge upwards as you insert it – the device has been designed to follow the natural curves inside your nose.


3

EN

When inside your nose, the arms of the device will wrap up and around the inside of your nostrils - as illustrated in images 3 and 4B.

4A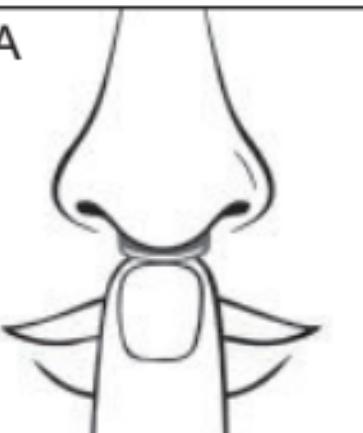


4B

EN

## ADJUSTING THE FIT:

To achieve optimal fit, you may need to expand or contract the loops by gently clicking the paddles outwards with your fingers as illustrated, which will activate the ratchet mechanism.

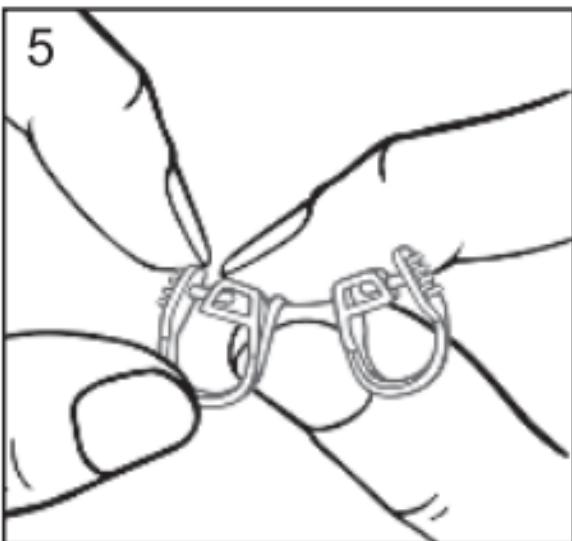## ATTENTION

EN

- The device is for personal use only and should not be shared with other people.
- If the device is removed from the packaging and not used immediately, rinse the device before use.
- Immediately after use, wash the device in warm water with mild soap paying close attention to the ratchet mechanism. Rinse the device thoroughly under running water and dry well before storing at room temperature, out of direct sunlight.
- Thoroughly rinse again prior to use.
- Be gentle when dilating the paddles and do not dilate beyond the pin length.
- Do not use the device if it is damaged in anyway.
- Do not intentionally break the bridge section and use one half of the device on its own.
- The device is not suitable for use by children under the age of 12.
- The device is intended to be used inside the nose only and may be up to 10 times.
- Do not use for more than 12 hours a day.
- If irritation occurs, discontinue use and consult your doctor.
- Do not use during contact sports or water sports or while swimming.
- Keep out reach of children.

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### CUSTOMER SERVICE:

For regional customer service details, please visit:

[mutesnoring.com/contact-us/](http://mutesnoring.com/contact-us/)

Designed in Australia. Made in China. Sold all over the world.

### PATENT PENDING

MUTE is a trademark of Rhinomed Limited.

ARW-11

### Description of Symbols:

	Manufacturer
	Batch Code
	Authorised EU Representative
	Consult Instructions for Use
	Not made with natural rubber latex
	Non Sterile
	Keep Dry
	Avoid direct sunlight
	Medical Device

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MDSS GmbH